

# Genenositcs

ANTIGEN COV-19 DETECTION TEST



**SANTANA MEDICARE**  
Health Solutions For A Better Tomorrow

[www.santanamedicare.com](http://www.santanamedicare.com)  
[sales@santanamedicare.com](mailto:sales@santanamedicare.com)  
phone: +27 87 809 3377

Sahpra License Number: 00001483MD



# Genenositics

## Product Name:

Novel Coronavirus (2019-nCoV)  
Antigen Rapid Test

## Overview


COVID-19 is caused by the virus severe acute respiratory syndrome coronavirus 2 (SARS-CoV-2) and is primarily transmitted from person-to-person through respiratory droplets. The estimated incubation period for COVID-19 is up to 14 days from the time of exposure, with a median incubation period of 4 to 5 days.

Antigen is generally detectable in upper respiratory specimens during the acute phase of infection. This rapid test represents a valuable alternative in the context of a global shortage of diagnostic tests and allows to obtain results quickly and reliably.



## Advantage of Antigen Testing

- Early detection
- Simple operation
- Fast result within 15 minute
- Detection without any equipment

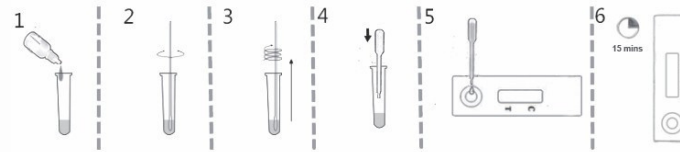

## Test Procedure:

1. Add 15 drops of the buffer into the extraction tube.
2. Insert the specimen collection swab into the buffer and rotate the swab constantly. Repeat several times and process for 2 minutes.
3. Squeeze the swab on the tube wall so that the liquid is extruded continuously. Take out the sample and discard the swab.
4. Use the pipette to fully suck the sample treatment fluid.
5. Dispense 3 drops from the extraction tube into the sample well of the tested card.
6. Wait 15 minutes to interpret and record the test result.

## Intended Use:

This kit is used for in vitro qualitative detection of novel coronavirus (2019-CoV) antigens in human throat swabs and nasal swab samples and is used for clinical auxiliary diagnosis of novel coronavirus infection. It cannot be used as a basis for diagnosis and exclusion of new coronavirus infection.

## Storage and Validity

Store at 2 ~ 30 °C, the validity period is tentatively 18 months.  
After opening the test card, it should be used as soon as possible within 1 hour. The sample treatment solution should be capped immediately after opening, and placed in a cool place, away from direct sunlight.  
See the product label for the production date and duration of use.

## Packing Specifications

25 tests/kit  
50 tests/kit



Imported and Distributed  
by Santana Medicare PTY  
Ltd.  
[www.santanamedicare.com](http://www.santanamedicare.com)  
[sales@santanamedicare.com](mailto:sales@santanamedicare.com)  
phone: +27 87 809 3377